

THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF VINCENT MEDICAL HOLDINGS LIMITED ANY REPRODUCTION IN PART OR WHOLE WITHOUT THE WRITTEN PERMISSION OF VINCENT MEDICAL HOLDINGS LIMITED PROHIBITED.

REV	EFFECTIVE DATE	ECO# / ECN#	DESCRIPTION	APPROVED
A	2021-01-25	N/A	The first release	Chevy xu
B	2021-04-12	HECN2100036	更新LOGO, 3图中的管路更新为6pin管路	Chevy xu
C		HECN2400092	更新内容	Jefferson Yu

此文件为：VUN-001呼吸湿化治疗仪快速操作指引技术资料。

Fast operation guide of VUN-001 O2FLO Respiratory Unit user manual technical information.

1. 尺寸：210x145mm (公差±5mm)

Size: 210x145mm (tolerance ±5mm)

2. 材质：250g双铜

Material: 250g double copper paper

3. 印刷：双面印刷

Printing: double-sided printing

4. 印刷颜色：4C (CMYK)

Printing Color: 4C (CMYK)

5. 要求双面过塑，左上角打孔，孔直径6mm，孔圈串链一条；

Double-sided plastic, make a hole on the top left corner and hole diameter 6mm, hole ring a string of chain.

6. 印刷内容需清晰无误；外观应整洁，无明显划痕、批锋、毛刺和气泡，文字和标识应清晰、准确、牢固。

Printed content must be clear and correct.Label should have clean surface without scratch、burrs、rag and bubble , symbols and characters should be clear 、accurate and stable on the label.

UNLESS OTHERWISE SPECIFIED
DIMENSIONS ARE IN _____
TOLERANCES ARE:
DECIMALS ANGLES
X ± X ± °
. X ± . X ± °
. XX ± . XX ± °
. XXX ± .

MATERIAL:

FINISH:

Designer:
设计

Check:
审核

Production:
生产

Regulations:
法规

Marketing:
市场

Approved By:
批准



VINCENT MEDICAL (DONGGUAN) TECHNOLOGY CO., LTD

TITLE: 快速指引

PART NO. 10026917


SCOPE : 呼吸湿化治疗仪\VUN-001

DWG. NO: A10026917REV: C

SCALE : N/A

SIZE: A4






SHEET : 1 OF 3

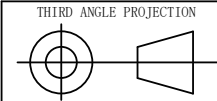

inspired O2FLO™

VUN-001

CN

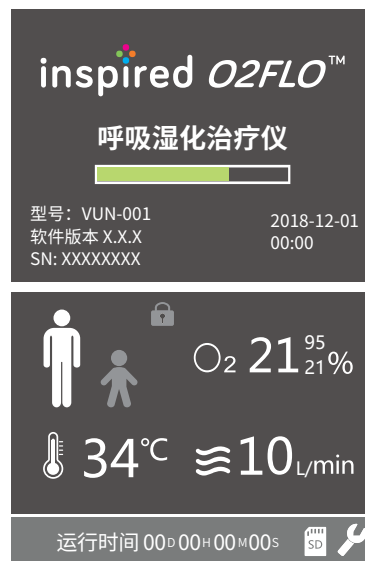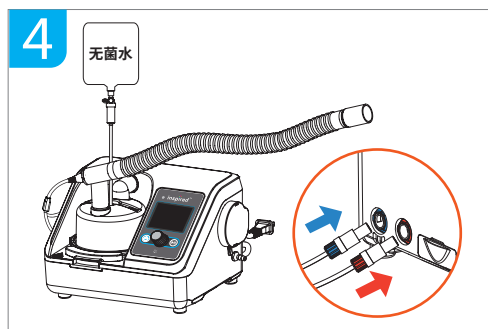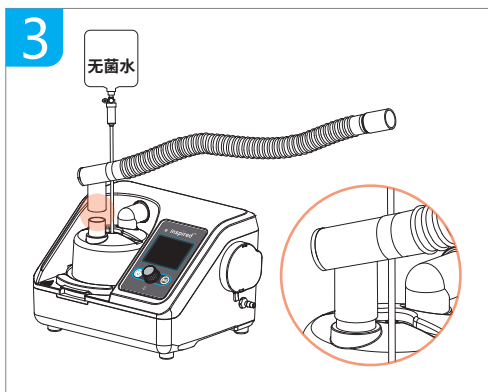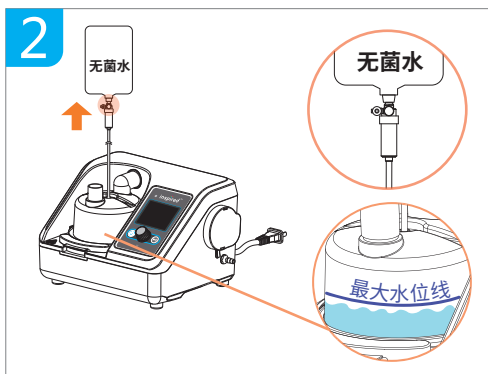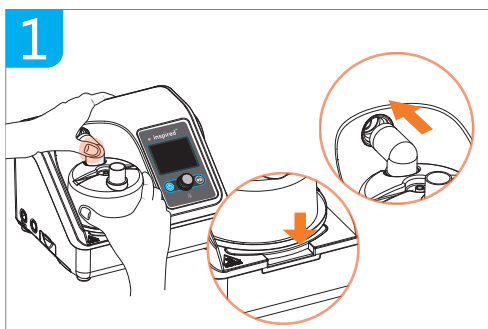

禁止使用
自来水



禁止使用
生理盐水



请勿往湿化罐
出气口倒水



请勿倾斜设备



设备暂停或未
运行时, 请确保
氧气源已关闭



设备暂停或
未运行时,
请勿供水



禁止使用消毒水
喷洒设备消毒

6 开始运行

↓

运行时间 00:00:00:05s

7

患者端界面	型号规格	露点温度 (°C)	流量 (L/min)
成人鼻导管	VANC-01 (S)	31-37	10-50
	VANC-02 (M)	31-37	10-60
	VANC-03 (L)	31-37	10-60
婴儿鼻导管	VINC-01 (S)	31-34	2-8
	VINC-02 (M)	31-34	2-8
	VINC-03 (L)	31-34	2-10
	VINC-04 (XL)	31-34	2-10

8

警告:

- 连接氧气源前请确保设备正在运行。
- 请勿使用附有加湿瓶的流量计。使用这种类型的流量计可能会导致性能受损、设备损坏和/或患者受伤。

9

警告:

请缓慢调节氧气流量, 避免损坏设备;
设备未使用时, 请确保氧气源已关闭。

注意: 设备不连接氧气供应时, 请用盖子盖上氧气进气口。

禁止使用
自来水

禁止使用
生理盐水

请勿往湿化罐
出气口倒水

请勿倾斜设备

设备暂停或未
运行时, 请确保
氧气源已关闭

设备暂停或
未运行时,
请勿供水

禁止使用消毒水
喷洒设备消毒